

FUJI DRI-CHEM CLEAN TIPS

[Warnings and precautions]

1. Do not reuse.
2. Do not touch tip-ends. It may cause damage to water repellent.
3. Do not contact tip-ends with other devices. It may cause contamination of water repellent.
4. Do not use a tip that fell on the floor as it could be contaminated.

[Intended use]

This product is used when pipetting samples such as whole blood, plasma, serum and control serum for FUJI DRI-CHEM system.

[Limitations of use]

This product is specially designed for FUJI DRI-CHEM system. This product cannot be applied to other systems.

[Composition]

Tube : Polypropylene

Water-repellent : Silicon resin

[Procedure]

Consult "INSTRUCTION MANUAL" for FUJI DRI-CHEM ANALYZER.

[Disposal information]

Used tips are categorized as infectious waste. Make sure to dispose of them in accordance with the Waste Disposal Law and other related regulations, which prescribe the proper method of disposal, such as incineration, melting, sterilization or disinfection.

[Storage and handling]

1. Do not store at high temperature to avoid deformation.
2. This product does not allow autoclave treatment before use.

[Contents]

96tips×6



Consult "INSTRUCTION MANUAL" for FUJI DRI-CHEM ANALYZER or at <http://www.fujifilm.com/products/medical/>



FUJIFILM Europe GmbH
Heesenstrasse 31, 40549 Düsseldorf, GERMANY



FUJIFILM Corporation
26-30, Nishiazabu 2-Chome, Minato-ku, Tokyo 106-8620, JAPAN



Do not reuse



Lot number



Manufacturer



Authorized representative in the European Community



Contains sufficient for <n> tests



In vitro diagnostic medical devices



Consult instructions for use