

## FUJI PLAIN TUBE

### [Warnings and precautions]

1. Do not reuse.
2. Do not use a deformed tube to avoid causing malfunction of the analyzer.
3. Do not use a tube that fell on the floor as it could be contaminated.

### [Intended use]

This product is used when collecting serum from whole blood.

### [Limitations of use]

This product is specially designed for FUJI DRI-CHEM system. This product cannot be applied to other systems.

### [Composition]

Tube, Cap : Polypropylene

### [Procedure]

Consult "INSTRUCTION MANUAL" for FUJI DRI-CHEM ANALYZER.

### [Disposal information]

Used tubes are categorized as infectious waste. Make sure to dispose of them in accordance with the Waste Disposal Law and other related regulations, which prescribe the proper method of disposal, such as incineration, melting, sterilization or disinfection.

### [Storage and handling]

1. Do not store at high temperature to avoid deformation.
2. Be sure to close the package after use to avoid contamination.

### [Contents]

1.5mL tube or 0.5mL tube : 500

Cap (red) : 500



Consult "INSTRUCTION MANUAL" for FUJI DRI-CHEM ANALYZER or at <http://www.fujifilm.com/products/medical/>



FUJIFILM Europe GmbH  
Heesenstrasse 31, 40549 Düsseldorf, GERMANY



FUJIFILM Corporation  
26-30, Nishiazabu 2-Chome, Minato-ku, Tokyo 106-8620, JAPAN



Do not reuse



Lot number



Manufacturer



Authorized representative in the European Community



Contains sufficient for <n> tests



In vitro diagnostic medical devices



Consult instructions for use